



THERMOSKIN SPRAY UNIT

2025 MODEL



INSTRUCTIONS FOR USE:

- 1 Unscrew cap of Thermoskin product.
- 2 Take dip tube out of cover.
- 3 Insert dip tube into Thermoskin product.
- 4 Screw cap down when dip tube is inserted.
- 5 Connect the electrical lead to the power point.
- 6 Turn spray control switch (6a) toward user to auto intermittent position. This controls the sensor (6b).
- 7 12v self-priming pump (7a). Turn power on at top of pump. When using for FIRST TIME ONLY, prime with rubber primer pump (7b). Turn power off when changing drums.
- 8 Use pressure regulator to vary amount of product sprayed if required. 50mls recommended dose.
- 9 CLEAN THE UNIT DAILY
When it is no longer required for the day:
 - Turn power off.
 - Remove the dip tube from the Thermoskin product and insert it into a drum of methylated spirits.
 - Turn spray control switch away from user to manual continuous position and disable the sensor.
 - Turn power on to allow any residual product to be flushed through (approx 30 seconds).
- 10 Once the cleaning procedure is complete, turn power off and put dip tube back in its cover to protect from dirt and dust.
- 11 Remove electrical lead.

STORE THERMOSKIN PRODUCT at ROOM TEMPERATURE
Keep the Product at room temperature or above 6°C.



HENRY KING

TEL: 02 9534 7455 | MOB: 0418 641 645

EMAIL: henry@thermoskin.com.au | WEB: www.thermoskin.com.au